
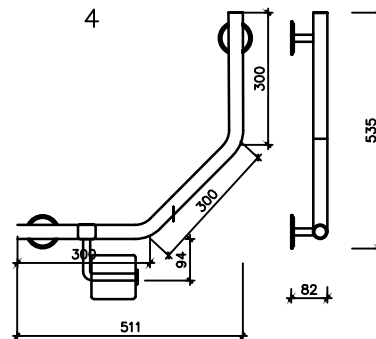
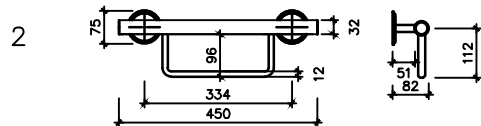


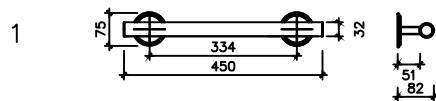

AMORE SUPPORT RAILS:

Material:

- Material : Solid brass (32mm)
- Colour : Available in 6 colours

- 1 Support rail
- 2 Support & towel rail
- 3 Liftable rail
- 4 Cranked rail with toilet roll holder
- 5 Shower column
- 6 Cistern rail

FIXATION COMPONENTS:



1. Mounting bracket



2. Cover plate



3. 8 x 65mm wall plug (x6)



4. 5 x 70mm full thread screws (x6)



5. 3mm allen key

Amore Support Rail – Mounting Instructions

Important: These rails must only be installed into solid brick or concrete walls. Installation into drywall or hollow partitions is not suitable for load-bearing safety rails.

Step-by-Step Installation Guide:

Determine Rail Position:

Hold the rail in the desired position on the wall. Ensure the wall brackets are facing upward, with the three screw holes at the top.

Mark Bracket Locations:

Use a pencil to mark the position of each bracket on the wall.

Use a spirit level to ensure both brackets are level before proceeding.

Drill Mounting Holes:

At each marked location, drill holes appropriate for solid brick.

Insert wall plugs into the holes.

Align the wall brackets with the drilled holes & fasten each bracket to the wall using the provided screws.

Ensure the brackets are firmly secured and level.

Attach the Back Plates:

Screw the back plate covers onto the mounted brackets.




Mount the Rail:

Slot the rail into position on the brackets. Using the provided Allen key, tighten the grub screws to secure the rail in place.

Final Check:

Ensure all fittings are tight and secure. Confirm the rail does not move or wiggle and is safe to use.

3 We are currently refining the design of the liftable rail. For enquiries, please reach out to us info@vanitihouse.com

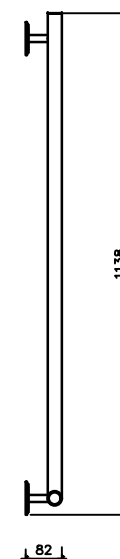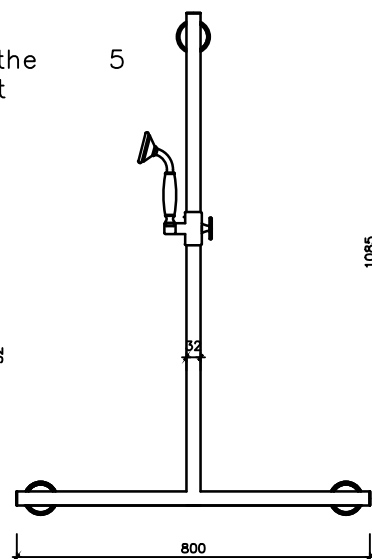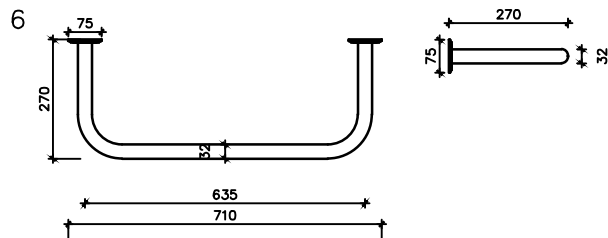

VANITI HOUSE – AMORE SUPPORT RAILS DESIGN DETAIL